

Vivago FIDO™ brings additional safety for nurses and other healthcare professionals working in a variety of institutions and units. Vivago FIDO™ can be used also as a personal alarm device for customers or patients in institutions or home like environments.

Use of the Vivago FIDO™ requires radio connection to a Vivago POINT base station and an active communication link to Vivago Vista server.


## Easy-to-use Vivago FIDO™

With the Vivago FIDO™ alarm device, you can easily make a manual alarm by pressing the Vivago button. After the alarm, the red LED light appears on the device.

Vivago  $FIDO^{\mathbf{m}}$  has a hook clip to allow attaching it for example to your clothes.



## Vivago

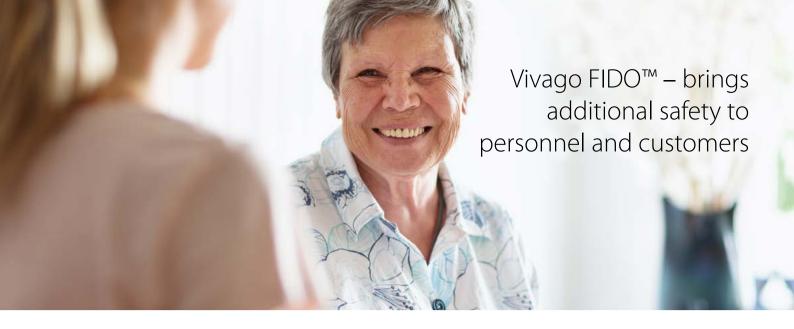

## **Vivago FIDO™ – Capabilities**

Push button for activation of a manual alarm

Vivago FIDO<sup>™</sup> can be configured to the use of a customer or healthcare professional Rechargeable battery

Automatic self-diagnostics

Notification of low battery and battery charge status

## Important!

- The device notifies the user about battery charge getting low by blinking the red LED once in every 16 seconds.
  A fully charged battery allows use for up to 12 months between re-charges.
- The device also notifies the system when the battery must be charged or when it is full.
- After the battery low notification, the battery must be charged within three days.
- Sales package includes Vivago FIDO™ personal safety device with a hook clip. The charger is not included in the default delivery and can be ordered separately. Chargers for Vivago watches are compatible with the Vivago FIDO™ alarm device.
- Vivago FIDO™ is waterproof, so you can keep it with you in the shower or in the bath. The device can't be used in hot conditions, such as Sauna.

