Vivago CARE watch

Quick Reference Guide

IF YOU THINK YOU NEED HELP, PLEASE PRESS THE CARE WATCH MANUAL ALARM BUTTON

IMMEDIATELY TO RAISE AN ALARM. PRESS THE BUTTON UNTIL THE RED LIGHT OF THE WATCH GOES ON.

DO NOT WAIT FOR THE SYSTEM TO DO AN ALARM FOR YOU AUTOMATICALLY.

• The CARE watch sends an automatic alarm e.g. in case of immobility within 30 minutes to 4 hours. It is possible to adjust the sensibility of the automatic alarm to be user-specific.

• After an alarm has been raised, the alarm recipient will be able to talk with you via the base unit's loudspeaker (Voice communication is a separate function that is used in some of the systems).

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Vivago

www.vivago.com

• Fasten the CARE watch so that it is in close contact with your skin at all times, but does not cause any discomfort or restrict your hand movement.

• CARE watch is waterproof; you may take a shower or a bath without removing the watch.

• Do not open the watch. Warranty is not valid if the watch has been opened.

IT IS IMPORTANT TO KEEP THE CARE WATCH ON YOUR WRIST AT ALL TIMES, EVEN AT NIGHT.

You should remove your CARE watch from your wrist in order to:

- Go to the sauna or other hot environments.

- Clean the CARE watch at least once a week according to the instructions in the User's Guide.

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www.vivago.com

IMPORTANT!

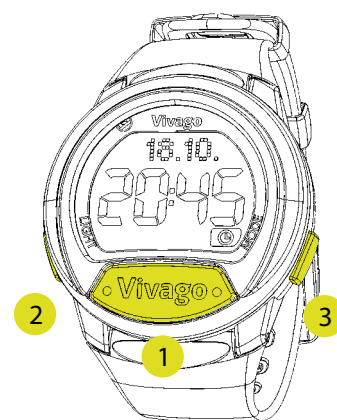
The CARE watch can only raise an alarm when you are within the reach of the Vivago base station.

The CARE watch can only send an automatic alarm when the watch is properly fastened around your wrist.

The CARE watch cannot send an automatic alarm when you have taken the watch off your wrist or if there is fabric or other material between the watch and your skin.

The CARE watch can only help you by sending an alarm, either manually or automatically. **DO NOT WAIT FOR THE SYSTEM TO RAISE AN ALARM FOR YOU AUTOMATICALLY IF YOU ARE ABLE TO DO IT YOURSELF.** Make sure you understand the purpose of this device and the limitation of the added safety it provides.

The CARE watch isn't a medical device. It shouldn't be used to monitor a specific medical condition.

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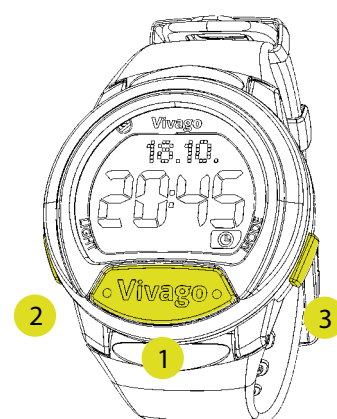
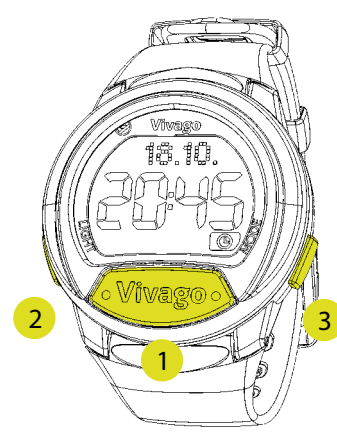
The button functions

CARE watch has three buttons:

1 VIVAGO button on the watch is meant for triggering the alarms, except when confirming changes in the setting mode of the watch.

2 A long press on LIGHT button will switch on the background light. The light will automatically switch off after 12 seconds.

3 MODE button is to be used when browsing through the different menus.

A long press of the MODE button or no press for 60 seconds will, from all menus, redirect the system to the watch mode.

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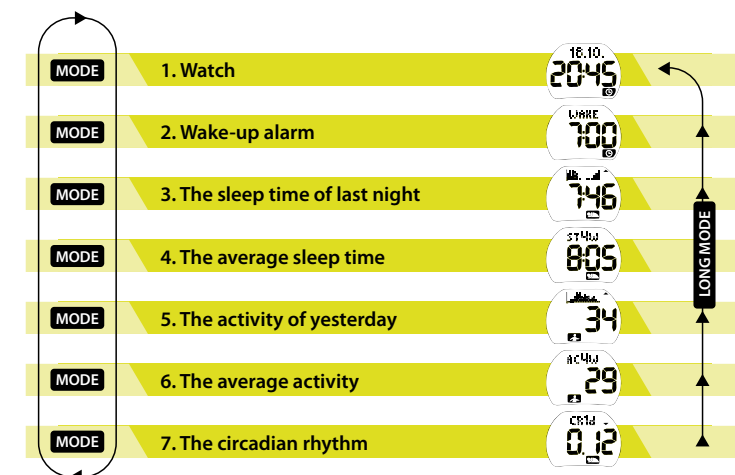
Menurakenne

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Settings

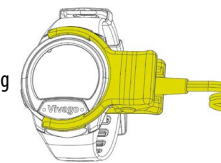
By pressing and holding both the LIGHT(2) and MODE(3) buttons simultaneously you enter the setting mode. The setting mode depends on in which display mode you enter the settings.

In the setting mode the buttons work as follows:

LIGHT: changes settings.
MODE: changes the numbers, for instance from hours to minutes when setting the time or changes the settings.
VIVAGO: confirms the setting and moves you to next set-up mode.

If you don't press any button for 60 seconds you will automatically return to the basic watch mode.

Please notice the correct positioning of the charger.

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