**Vivago**




## Vivago DOMI

Pikakäyttöohje – Snabbanvisning – Quick reference guide –  
Guide d'utilisation rapide – Kurzanleitung – Guida rapida –  
Beknopte handleiding – Kiirjuhend

# Vivago DOMI

Pikakäyttöohje – Snabbanvisning – Quick reference guide – Guide d'utilisation rapide – Kurzanleitung – Guida rapida – Beknopte handleiding – Kiirjuhend

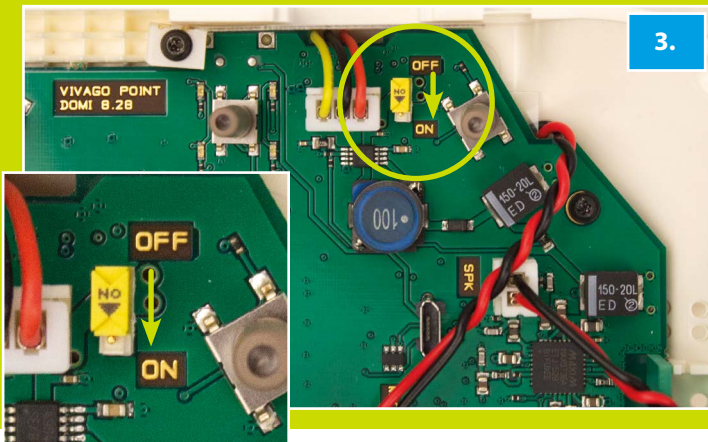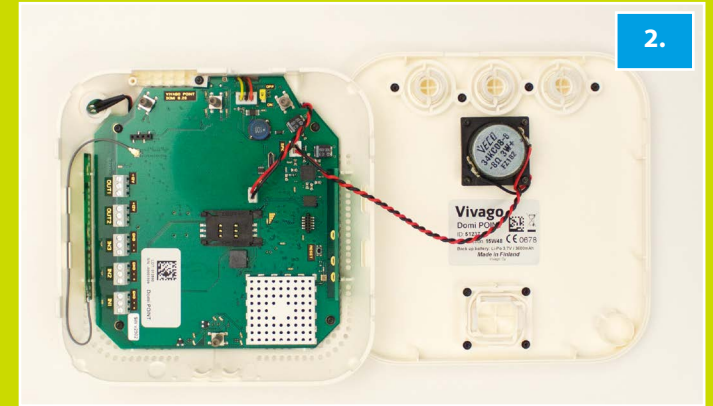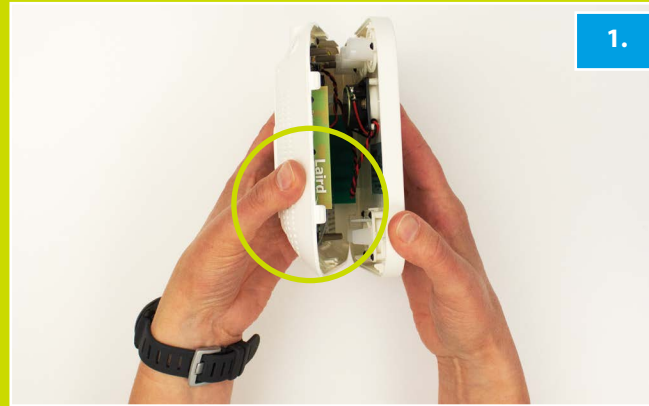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## A. What is in the box:

- Vivago CARE watch and charger
- Vivago Domi POINT care phone and power supply
- Quick reference guide: Note! Please also read the user's guide available at [guides.vivago.com](http://guides.vivago.com)

## Vivago DOMI – getting started

Perform the following steps before starting to use the device. Refer to the illustrations in this guide.

1. Open the cover of the care phone by pressing carefully on the side.
2. Place the cover and the underpart of the care phone side by side on an even surface as illustrated.
3. Move the yellow battery switch inside the care phone, near the top, to position ON. Close the cover carefully.
4. Carefully connect the micro-USB connector of the power supply to the care phone. Note! Make sure that the triangle on the connector is visible as in the image.
5. Plug the power supply into a wall socket.
6. The care phone is ready for use when only a solid blue indicator light is illuminated.
7. Charge the Vivago CARE watch for about 2 hours.
8. Do a test alarm with the watch by pressing the Vivago button until a red light in the watch lights up. You can also trigger an alarm by briefly pressing the red alarm button on the care phone. You can cancel the test alarm, or an alarm that is triggered by accident when the device is used, by pressing the STOP button on the care phone.

Note! Keep the device plugged into the mains electricity supply after setting it up for use. A warning will sound if the device is unplugged.

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