
Vivago

Vivago DOMI

Pikakäyttöohje – Snabbguide – Quick set up guide –
Guide de configuration rapide – Kurzanleitung – Guida rapida –
Beknopte handleiding – Lühijuhend – Gyors útmutató – Hurtigguide –
Guia de referência rápida – Guía rápida de consulta – Kratek priročnik

Vivago DOMI

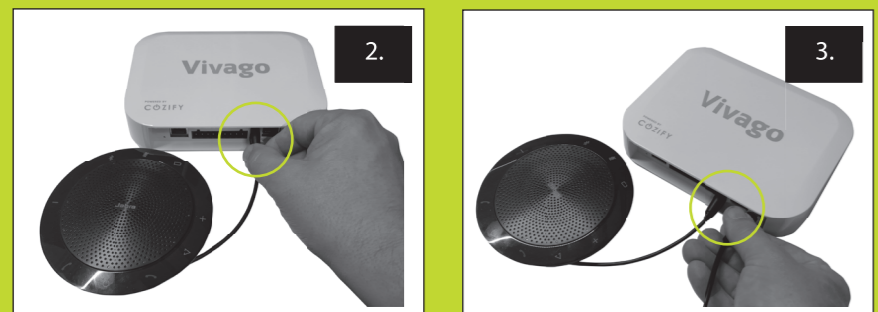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A. Contents of the package:

- Vivago watch and charger
- Vivago base station and power supply
- Speaker
- Control switch (optional)
- Quick reference guide

EN

Getting started

Perform the following steps:

- Place the base station and speaker on a table, for example.
- Locate the base station and speaker close to the power socket in a central location in the home to ensure good radio coverage.

- Charge the Vivago watch for approximately 2 hours.
- Connect the speaker's USB cable to the USB connector on the back of the base station.
- Connect the power supply to the power connector on the back of the base station.
- Plug the power supply into a power outlet. The base station flashes the indicator light for a few minutes during start-up and is ready for use when the green indicator light is continuously on at the back of the base station.
- There shall be two active light indicators on in the speaker. The Power On(1) and the Charge status indicator(2) lights. When charge status indicator light is red, the battery is being charged and when green the battery is charged. In case the Blue indicator light is on press the Bluetooth button(3) shortly to switch the light off. The speaker volume and microphone mute are controlled automatically and set to their defaults when a new phone call is established. Microphone is not working if the mute is activated. This is indicated by a series of red indicator lights circling around the device. To unmute press the mute button(4) once. Do not press the buttons unnecessarily.
- Generate a test alarm with the watch by pressing the Vivago button until a red light in the watch lights up. If you have a control switch, also generate an alarm by pressing the alarm button on the control switch.

Please note!

- Vivago DOMI is intended for indoor use at normal, dry indoor temperatures.
- Keep the device plugged into the mains electricity supply after setting it up for use.
- Do not open the device.

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Vivago® offers smart safety and wellbeing solutions for preventive care. The innovative solutions cover the whole care chain from home care to assisted living, hospitals and rehabilitation. Vivago solutions increase the feeling of safety with automatic alarms and by providing wide range of wellbeing information to support daily care and to follow-up the effectiveness of care. Real-time notifications inform about the changes in the person's wellbeing and it enables care providers to react proactively and focus on individual care.

Vivago

 <https://guides.vivago.com/>

Vivago Oy

Lars Sonckin kaari 14, FI-02600 Espoo, Finland

Tel. +358 10 2190 610 | info@vivago.com

www.vivago.com | www.vivago.fi